

Getinge Assured

Bowie-Dick Test Card TT

The Getinge Assured Bowie-Dick Test Card TT is for daily monitoring of small pre-vacuum type B steam sterilizers operating at 134°C. The card is specially packaged and designed to meet ISO 11140-6 sub-atmospheric cycle requirements.

This Getinge Assured product is tested and verified to have equivalent and approved performance at both ISO 11140-6 sub-atmospheric cycles and Getinge Bowie-Dick cycles for type B pre-vacuum table top sterilizers.



Performance Characteristics

If the sterilizer is functioning properly, the indicator graphic will show a uniform color change from purple to green throughout the entire indicator image. A brown or non-uniform green color may indicate problems with steam conditions in the sterilizer chamber other than non-condensable gases or residual air. This may require a sterilizer service call.

Physical Characteristics

- **Dimensions:** L 86 mm x W 36 mm
- **Packaging:** 1 ring and 15 test cards

Environmental Limitations

Storage: In provided sealed packaging at room temperature 10°-38°C and 10-60% Relative Humidity. Protect from moisture and excess humidity.

Standards Classification

24 months. The expiry date is printed on the product packaging.

Standards Classification

ISO 11140-6 sub-atmospheric cycle

Ordering Information

Getinge Assured	Ordering number
Bowie-Dick Test Card TT	6005500588