

Clinicare

Alcohol Free

Hospital Grade - Disinfectant Wipes

Kills
COVID-19
in **30**
seconds



Multi-Purpose

- For use on all hard surfaces
- Suitable for use on plastics

Tested under dirty conditions for proven and fast activity

- Only 30 seconds wait time for activity against Covid-19

The most effective wipe in the market

- The use of specially made wipes that has a uniformed flat surface gives the wipes exceptional characteristics for removing soiling and decontaminating the surface. When pre-saturated with Clinicare Alcohol Free Solution the wipe offers a uniform application and has great surface coverage

Peace of mind

- Broad Spectrum multi-purpose formula

Versatile

- Acts as a decontaminating and disinfectant wipe replacing the need for different types of wipes and solutions for surface cleaning

**BACTERICIDAL
YEASTICIDAL
SARS-CoV-2
(COVID-19)**



**Effective Against (kill time)
TESTED UNDER DIRTY CONDITIONS**

Pseudomonas aeruginosa (1 min)
Escherichia coli (1 min)
Staphylococcus aureus (3 mins)
Methicillin Resistant Staphylococcus aureus (MRSA) (1 min)
Enterococcus hirae (1 min)
SARS-CoV-2 (COVID-19) (30 secs)
Candida albicans (1 min)

Clinicare Alcohol Free Hospital Grade - Disinfectant Wipes


STEP 1: Decontaminate



A. Dispense Clinicare towelette



B: Wiping from clean to dirty in an S-shaped pattern, ensuring not to go over the same area twice

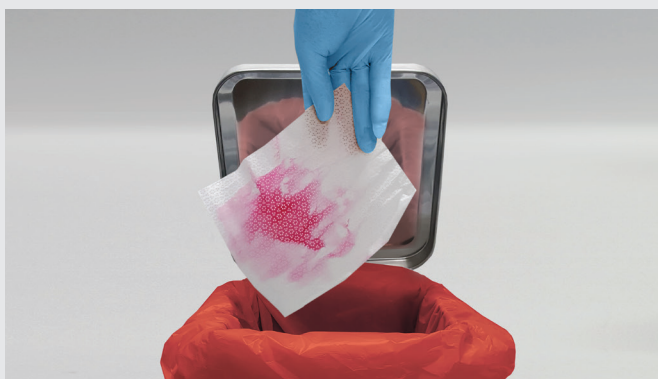

C. Dispose of used towelette

STEP 2: Disinfect



A. Dispense another Clinicare towelette



B: Wiping the decontaminated area from top to bottom in an S-shaped pattern, ensuring not to go over the same area twice.



C. Dispose of used towelette

DL2320
DL2320R

Clinicare Alcohol Free Canister (180)
Clinicare Alcohol Free Refill (180)



AUSTRALIAN MADE
AUST L: 373700

Dentalife
AUSTRALIA

Manufactured by:
Dentalife Australia Pty Ltd
Factory 9/505 Maroondah Highway
Ringwood, Victoria 3134 Australia
Phone (03) 9879 1226
www.dentalife.com.au