Jincare Alcohol Free

Instrument Grade - Low Level Disinfectant Wipes



Multi-Purpose

- For use on all hard surfaces
- Suitable for use on plastics

Tested under dirty conditions for proven and fast activity

- Only 30 seconds wait time for activity against Covid-19 The most effective wipe in the market
- Clinicare Alcohol Free Instrument Grade Low Level Disinfectant Wipes is a highly effective cleaner and disinfectant. It has been developed for use in hospital operating rooms, isolation areas, neonatal units and other critical care areas where environmental control of cross contamination is important.

Peace of mind

- Broad Spectrum multi-purpose formula
- Acts as a decontaminating and disinfectant wipe replacing the need for different types of wipes and solutions for surface cleaning

DentaLife

Manufactured by:

Dentalife Australia Pty Ltd Factory 9/505 Maroondah Highway Ringwood, Victoria 3134 Australia Developed and manufactured in Australia www.dentalife.com.au



Effective Against (kill time) TESTED UNDER DIRTY CONDITIONS

Pseudomonas aeruginosa Escherichia coli Staphylococcus aureus Methicillin Resistant Staphylococcus aureus (MRSA) Enterococcus hirae Proteus vulgaris Klebsiella pneumoniae Klebsiella pneumoniae (CRE) Vancomycin Resistant Enterococcus faecium (VRE) Salmonella enterica Candida albicans SARS-CoV-2 (COVID-19) Vaccinia virus (Mpox) Herpes Simplex virus Type 1 Influenza virus (H1N1, H3N2)





Clinicare Alcohol Free

Instrument Grade - Low Level Disinfectant Wipes Alcohol Free formula

Active Ingredients	Benzethonium Chloride 0.40% w/v, Cetrimide 0.45% w/v, Polyaminopropyl biguanide 0.1% w/v
Wipe Composition	Clinicare Alcohol Free wipes are available in 180 Towelettes in canister and refill pouch – Meltblown 38 gsm (15 x 17 cm) 200 Towelettes in Flat pack packaging – Meltblown 30 gsm (18.5 x 26 cm)
Uses	Clinicare Alcohol Free Instrument Grade - Low Level Disinfectant Wipes is a highly effective cleaner and disinfectant. It has been developed for use in hospital operating rooms, isolation areas, neonatal units and other critical care areas where environmental control of cross contamination is important.
Advantages	 1 minute kill time BACTERICIDAL, 30 seconds kill SARS-CoV-2(COVID-19) Alcohol Free Broad Spectrum, fast acting 2 in 1 decontaminant cleaner and disinfectant Formulated and tested for use in healthcare and critical environments Safe for use on all surfaces including plastics, metals, ceramics, leather, vinyl, etc.
Active Against	Pseudomonas aeruginosa Escherichia coli Staphylococcus aureus Methicillin Resistant Staphylococcus aureus (MRSA) Enterococcus hirae Proteus vulgaris Klebsiella pneumoniae Klebsiella pneumoniae (CRE) Vancomycin Resistant Enterococcus faecium (VRE) Salmonella enterica Candida albicans SARS-CoV-2 (COVID-19) Vaccinia virus (Mpox) Herpes Simplex virus Type 1 Influenza virus (H1N1, H3N2)
Contraindications	Not to be mixed with any detergents, chemicals or other cleaning agents.
Precautions	Do not swallow. Avoid contact with skin and eyes. Use undiluted. If visible soiling is evident, surfaces should be pre-cleaned / decontaminated before disinfection.
Clinical Procedure	Decontaminate the surface using Clinicare Alcohol Free Instrument Grade - Low Level Disinfectant Wipes or Clinicare Neutral Detergent to reduce contamination and prepare the surface. Wipe the surface from clean to dirty in an S-shaped pattern, ensuring not to go over the same area twice. Dispose of all materials used for decontamination. To disinfect surfaces: Thoroughly wet the surface using Clinicare Alcohol Free Instrument Grade - Low Level Disinfectant Wipes. Wipe the decontaminated area from top to bottom in an S-shaped pattern. Leave the surface wet at room (ambient) temperature for the required activity time (kill time) as per the activity table on the label or above in the 'Active Against' section.
Poison Schedule	Not Applicable
Safety Directions	Avoid contact with eyes. Avoid contact with skin. Wash hands after use.
First Aid	For advice, contact a Poisons Information Centre (Australia) 13 11 26
Regulatory	ARTG 405881 Class IIb
Presentations	DL2371 Clinicare AF Instrument Grade - Low Level Disinfectant Wipes 200 Flat Pack DL2378 Clinicare AF Instrument Grade - Low Level Disinfectant Wipes 30 Flat Pack DL2321 Clinicare AF Instrument Grade - Low Level Disinfectant Wipes Canister (180)
Tresentations	DL2321R Clinicare AF Instrument Grade - Low Level Disinfectant Wipes Refill
Storage	DL2321R Clinicare AF Instrument Grade - Low Level Disinfectant Wipes Refill Store below 30 °C in a cool, dry place.